

Evidence to support automation in on-ear and remote hearing solutions.

Brandy Heckroodt, Au.D.

EAA 2023

1


Presenter disclosures

- Financial: I am employed by Phonak.
- Non-financial: I have no relevant non-financial relationships to disclose.

2

Challenges with manual programs


Figure 3. The percentage of participants scoring below satisfactory for each PHAST skill task. VC = volume control.

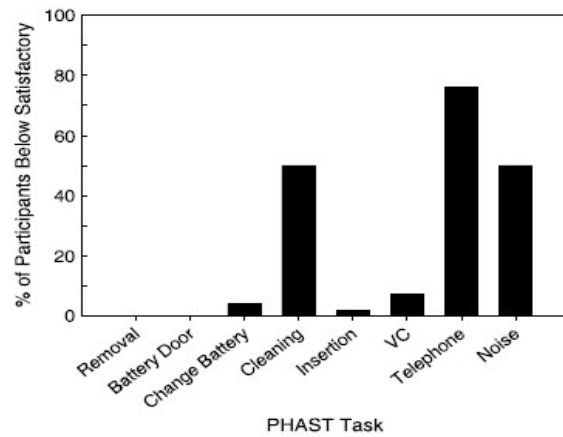

Doherty, Karen A., and Jamie L. Desjardins. "The practical hearing aids skills test--revised." *American Journal of Audiology*, vol. 21, no. 1, June 2012, pp. 100+. *Gale Academic OneFile*, link.gale.com/apps/doc/A293544496/AONE?u=maine_oweb&sid=googleScholar&xid=ca700790. Accessed 8 June 2023.

3

Manual switching for 24 children aged 11-17

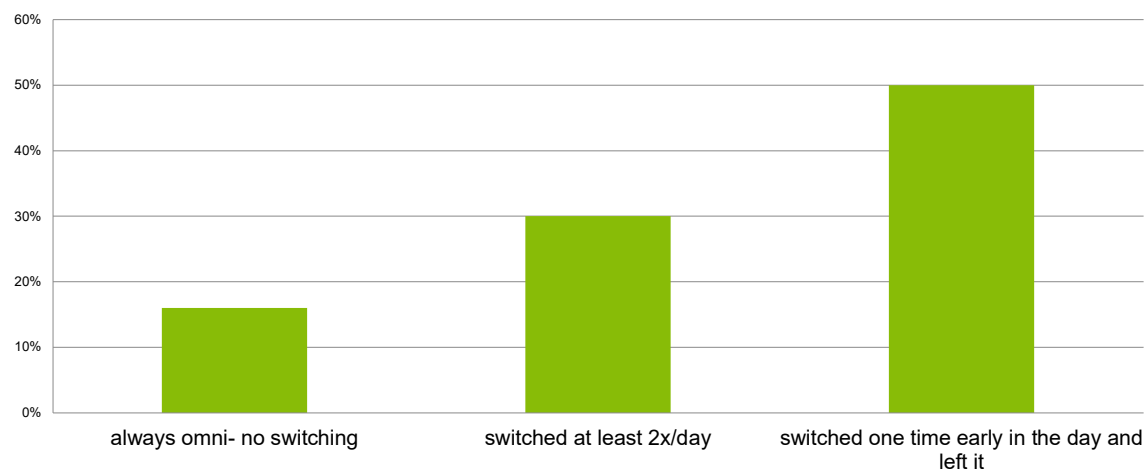


Ricketts, A Sound Foundation Through Early Amplification 2010

4

Schools are dynamic environments

Acoustics of a child's school day

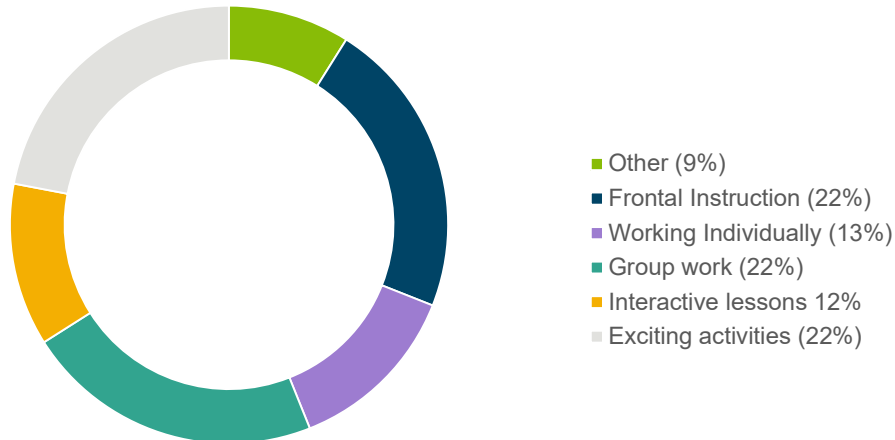


Fellner, M., Rich, S., & Jones, C. (2016) Automatic and directional for kids. Phonak Insight. retrieved from <https://www.phonak.com/en-us/professionals/audiology-hub/evidence-library>, retrieved June 8, 2023.

5

Schools are dynamic environments

Acoustics of a child's school day

- **Frontal Instruction**—hearing performance reported as “mostly satisfying”
- **Working individually**—not associated with listening problems
- **Group work**—unsatisfactory listening situation
- **Exciting activities**—class breaks, recess, sports, choosing partners for group work, getting ready for lunch, recess, specials, etc.

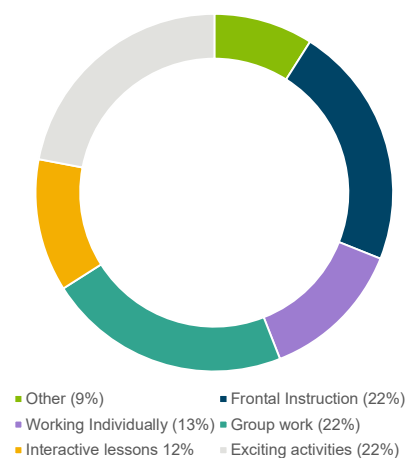

Fellner, M., Rich, S., & Jones, C. (2016) Automatic and directional for kids. Phonak Insight. retrieved from <https://www.phonak.com/en-us/professionals/audiology-hub/evidence-library>, retrieved June 8, 2023.

6

Schools are dynamic environments: Interviews with students

Group and interactive lessons are the most difficult to understand their peers.

That's 1/3 of a student's day where they continue to struggle to interact and understand well.

Exciting activities when other children are shouting or yelling causes discomfort.

Feliner, M., Rich, S., & Jones, C. (2016) Automatic and directional for kids. Phonak Insight. retrieved from <https://www.phonak.com/en-us/professionals/audiology-hub/evidence-library>, retrieved June 8, 2023.

7

What happens when children do not have access to audibility or are uncomfortable?

- Withdraw from activity
- Misunderstand instructions
- Listening fatigue
- Remove devices
- Listening break


8

The need for different microphone modes in school

Average number of switches needed per hour to maintain the appropriate microphone mode:

3.4


Range: 1 to 22

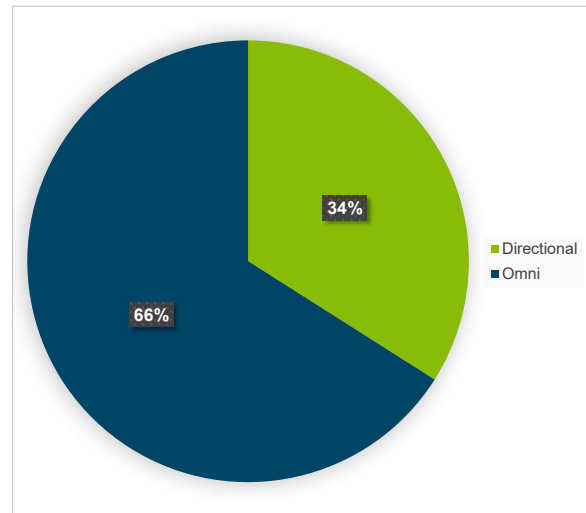


Ricketts T.A., Picou E.M., Galster J.A., Federman J., & Sladen D.P. (2010). Potential for Directional Hearing Aid Benefit in Classrooms: Field Data. A Sound Foundation Through Early Amplification: 143-152.

9


Monaural and binaural signal processing in hearing aids

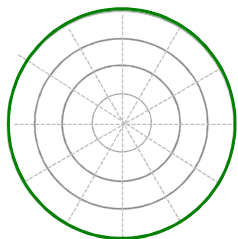


Omni

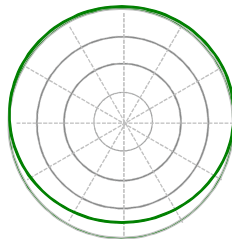

Real ear sound

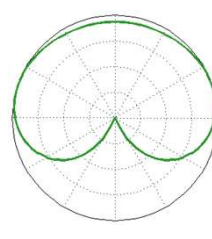


UltraZoom

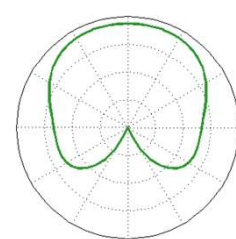

StereoZoom

10

Automation in hearing aids

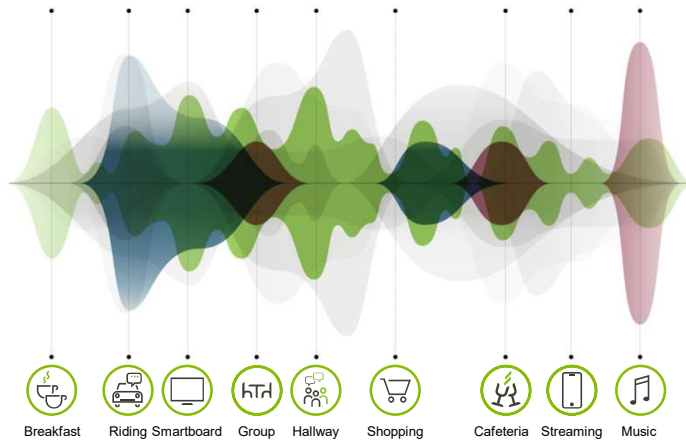

AutoSense Sky OS

Orchestrating speech intelligibility and comfort features

700 scans per second


for real-time adjustment to the environment

Improves comfort and speech understanding

Accurate and precise

11

Automation of AutoSense Sky OS

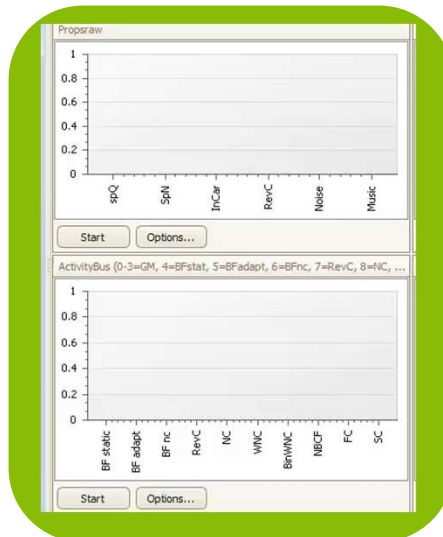


accurate detection

precision processing


12

AutoSense Sky OS: Automation for acoustic and streamed signals

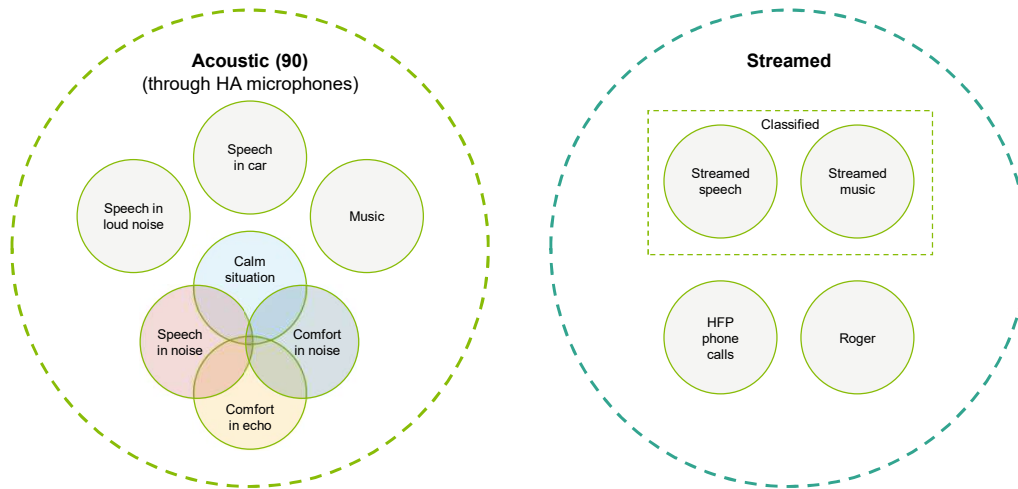


13

AutoSense Sky OS: Automation for acoustic and streamed signals

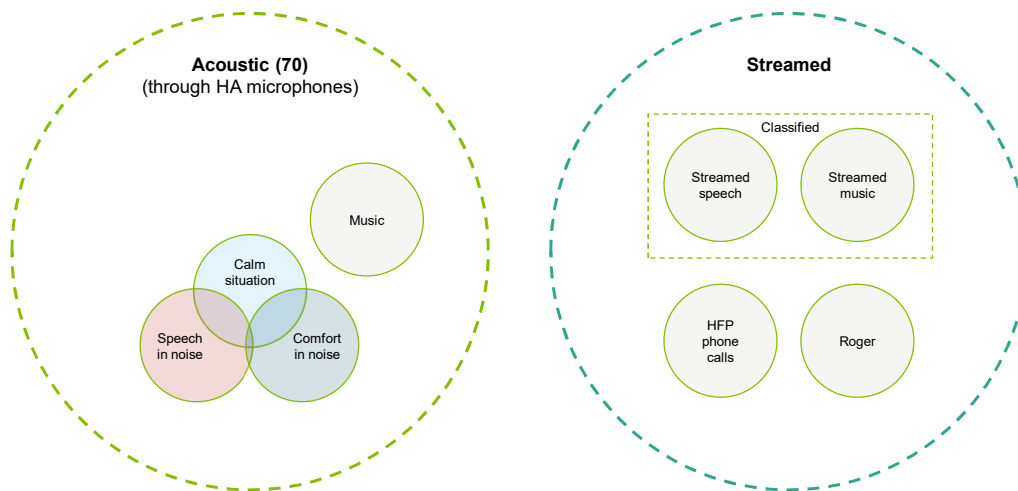


14

AutoSense Sky OS: Automation for acoustic and streamed signals

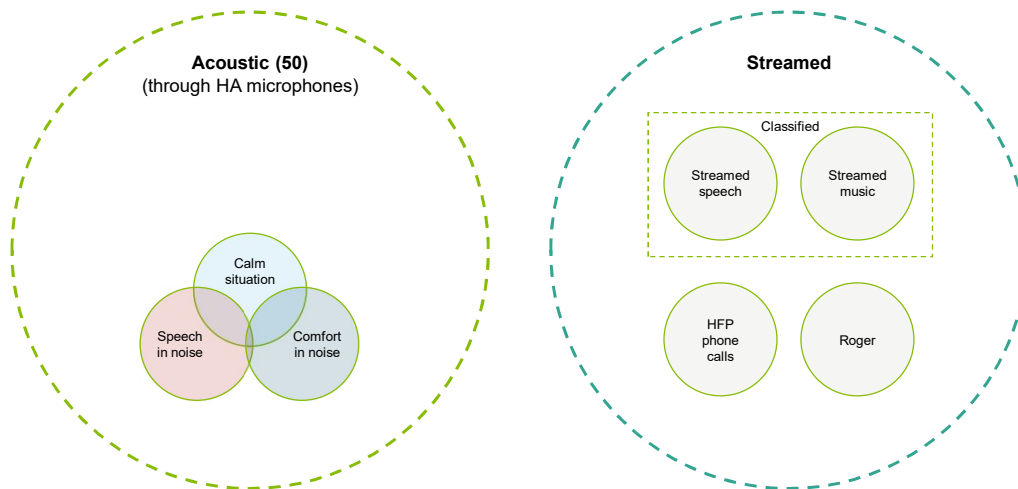

15

A look at automation via Target fitting software

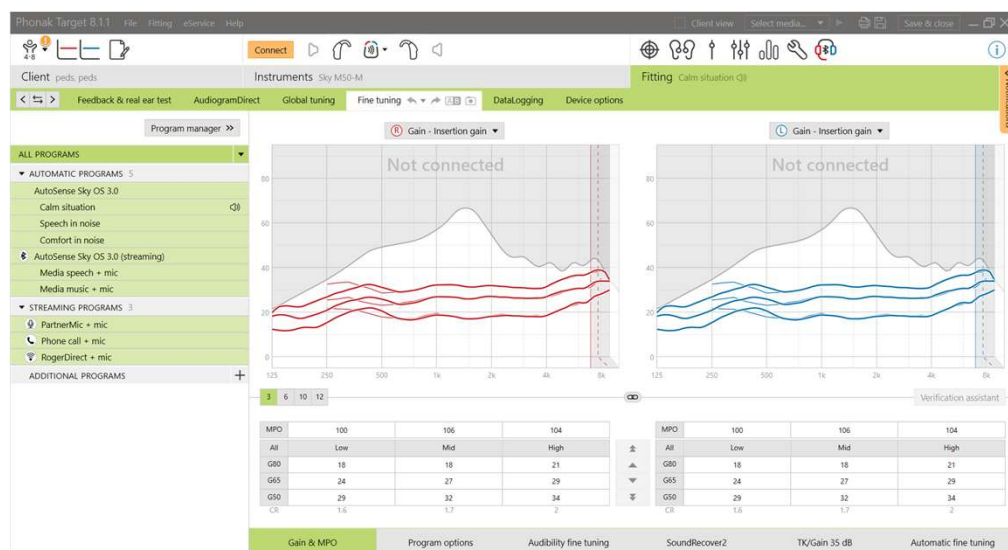

16

A day in the life

40%

of speech in a child's typical day comes from a distance of over six feet and/or with 'overlapping' noise



Mulla, I., & McCracken, W. (2014). Frequency modulation for preschoolers with hearing loss. *In Seminars in Hearing*, 35, (3), 206-216. Thieme Medical Publishers.


17

Automation of Roger




18

A look at automation via Target fitting software

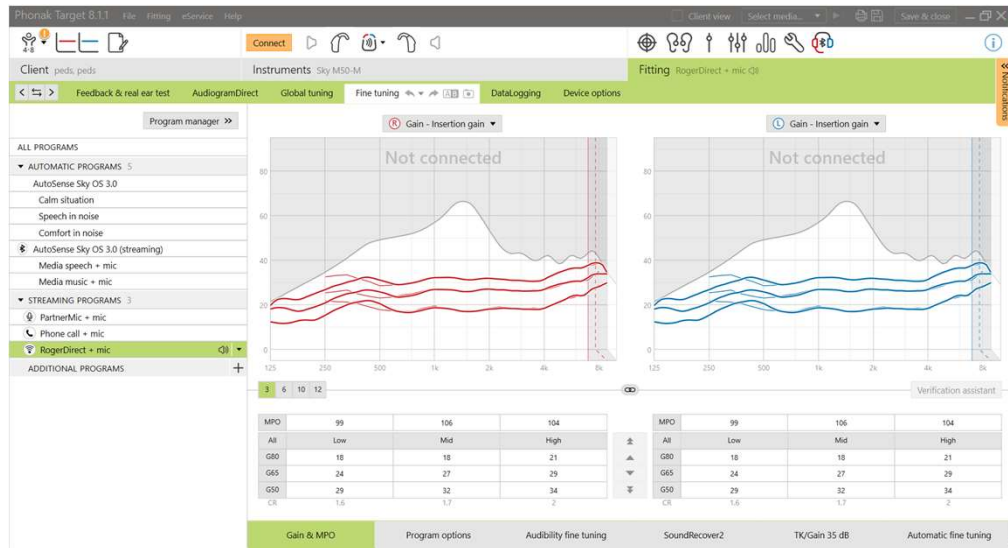


19

A look at automation via Target fitting software

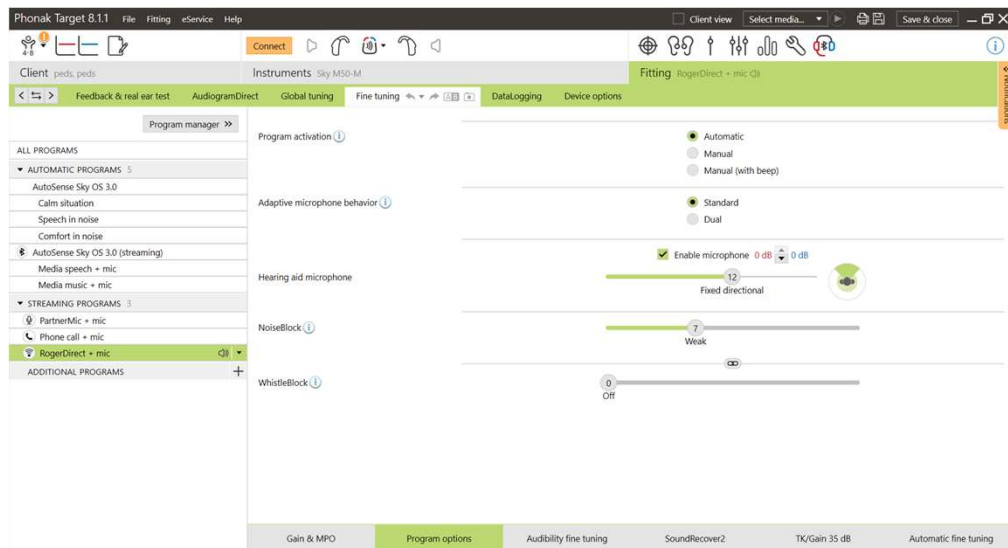


20

A look at automation via Target fitting software

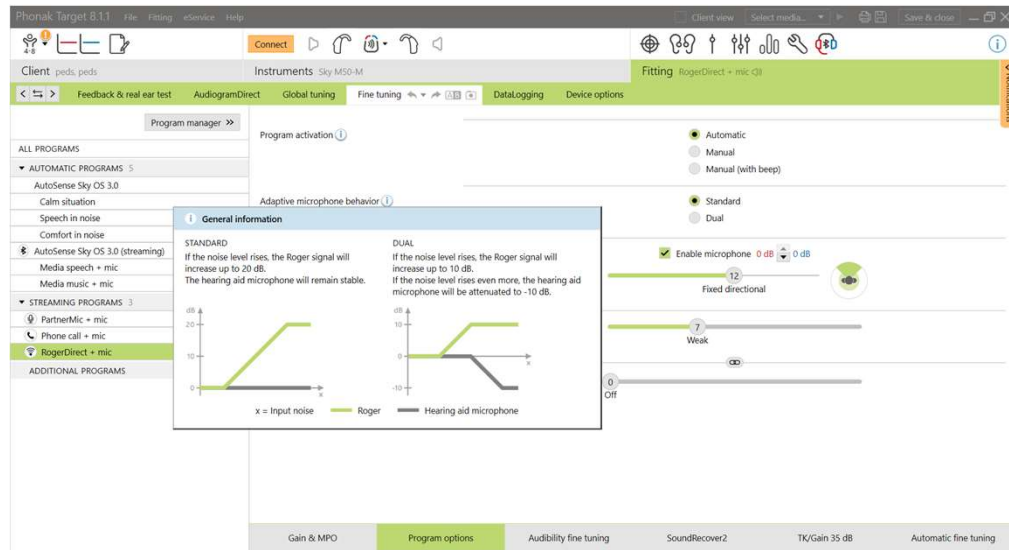

21

A look at automation via Target fitting software

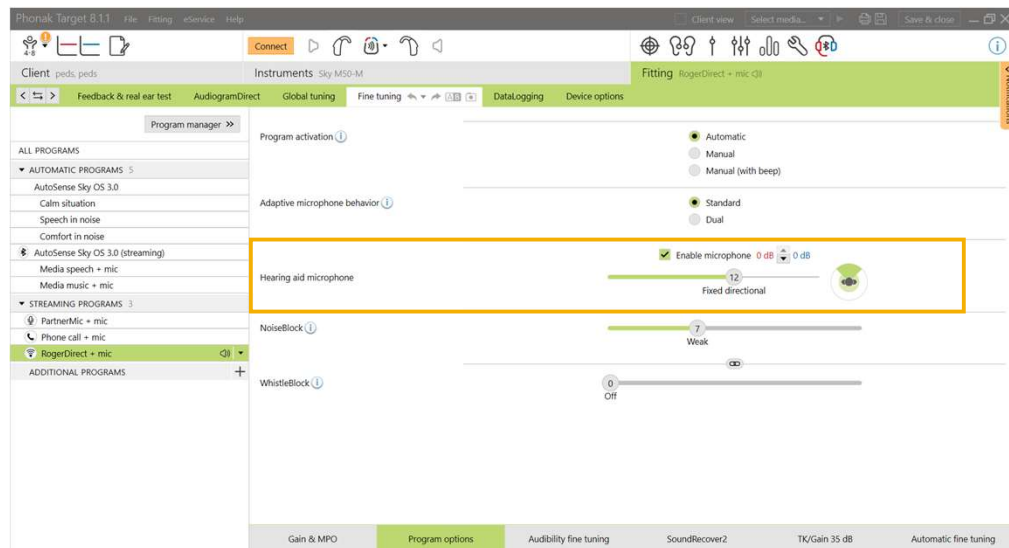

22

Roger+Directional

- Adaptively activated
 - Based on noise level and presence of speech
- Fixed directional beamformer
- Near-field benefit



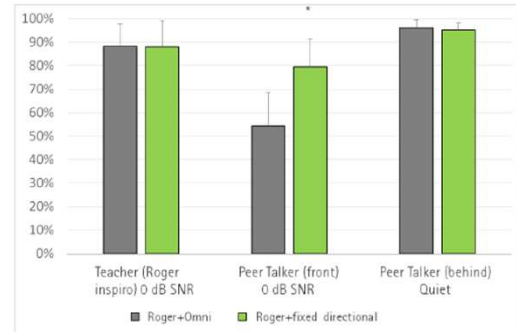


Figure 5. The average speech recognition scores for the Roger+Mic_{Omni} mode and Roger+Mic_{directional} modes in each of the three conditions.

23

Hearing aid automation can affect your Roger transparency measures

Activate Verification mode in Roger Touchscreen Mic
 [Settings], [Verification mode], [Activate]

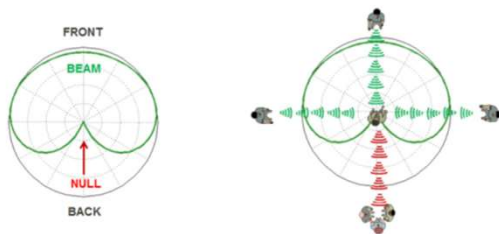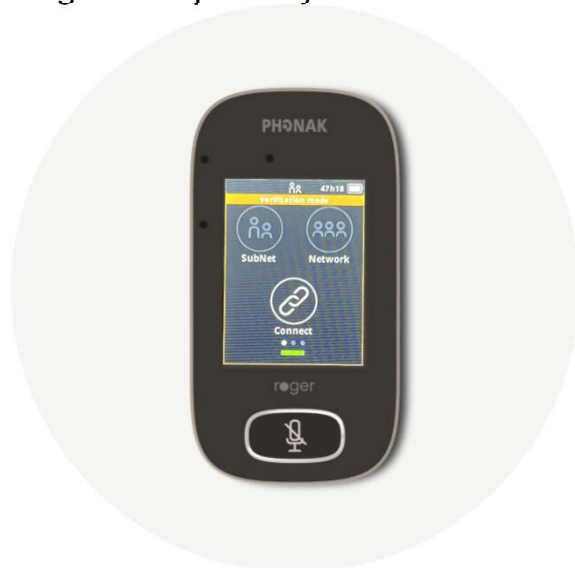

24

Hearing aid automation can affect your Roger transparency measures

Activate Verification mode in Roger Select




25

Hearing aid automation can affect your Roger transparency measures

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
Activate Verification mode in Roger On



26



While we're here, let's review how to adjust Easy Gain (Select and On)




27


Automation with Remote Mics



28

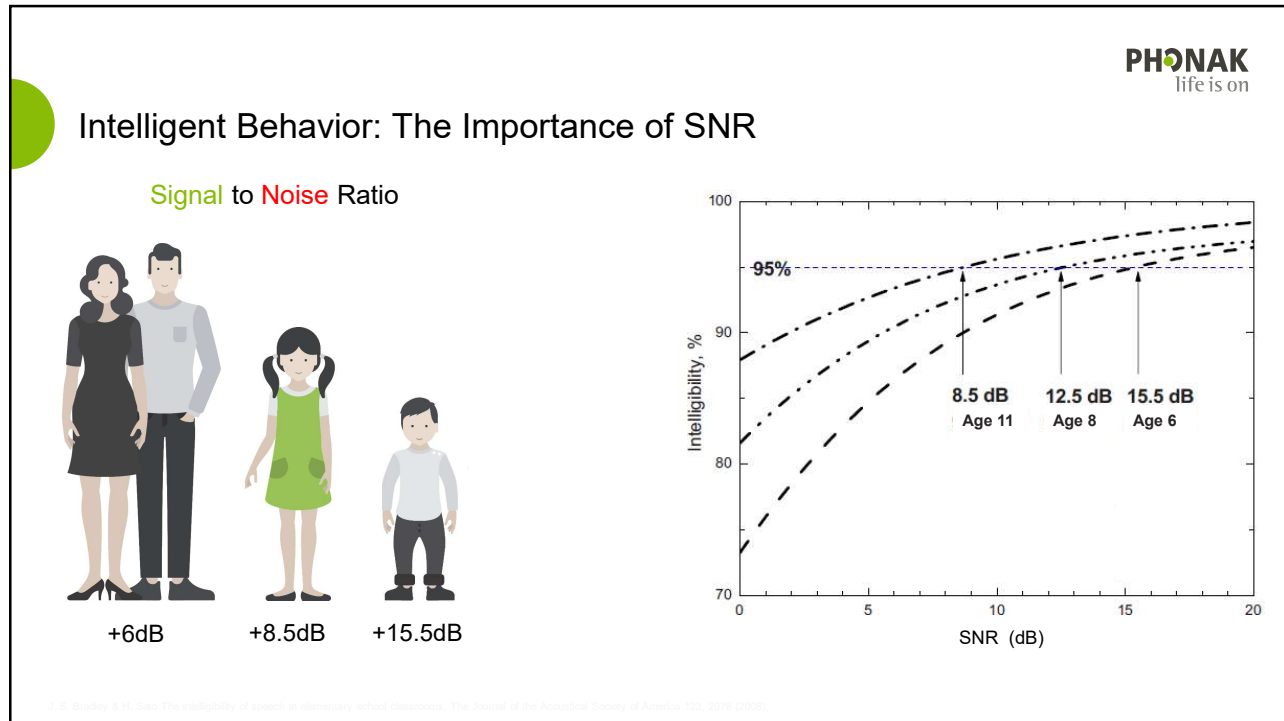


29

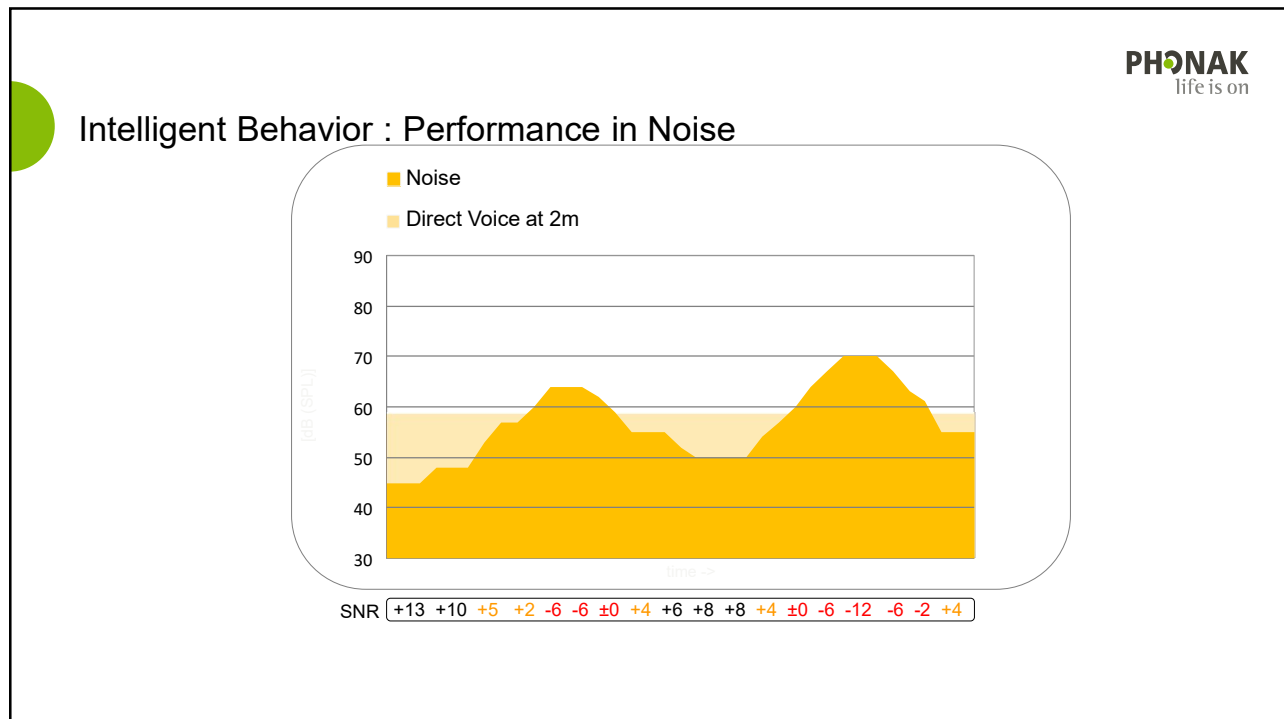


30

Intelligent Behavior : Performance in Noise

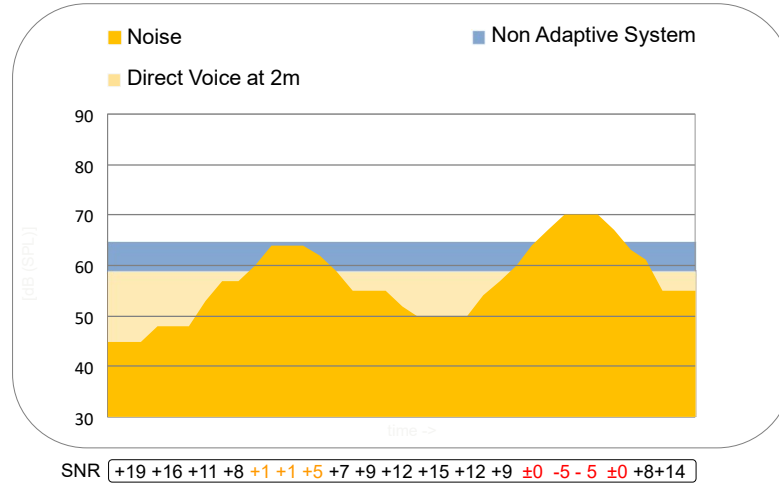


31

Intelligent Behavior : Performance in Noise

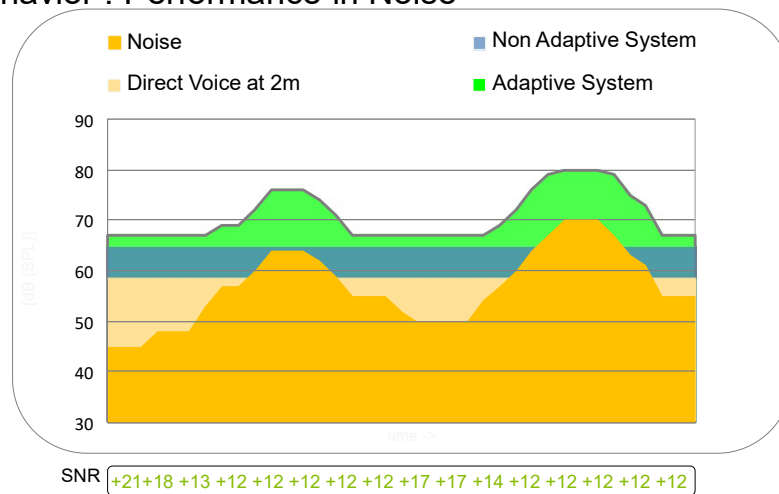

32

Intelligent Behavior : Signal Optimization

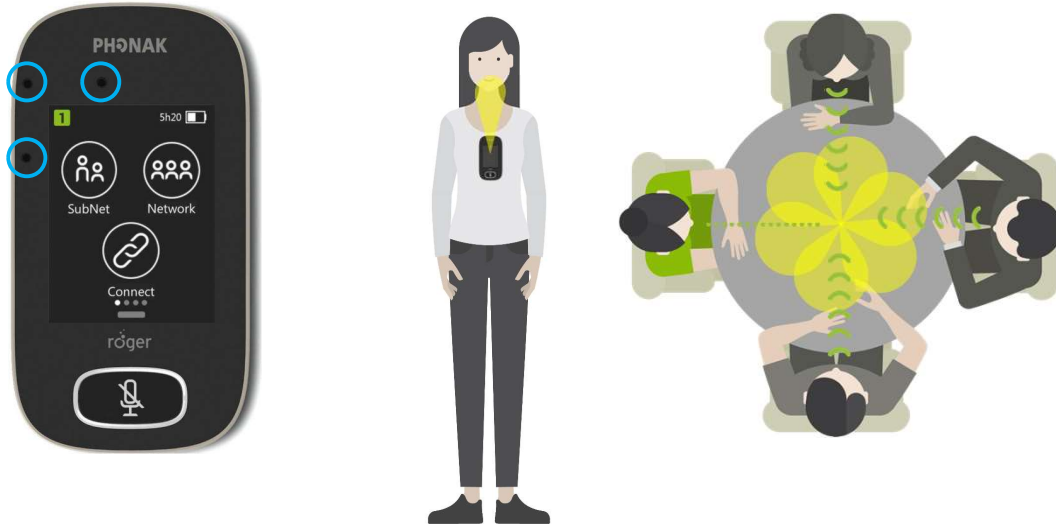

33

Intelligent Behavior : Signal Optimization




34

Intelligent Behavior : Voice Detection



35

Roger On/On iN

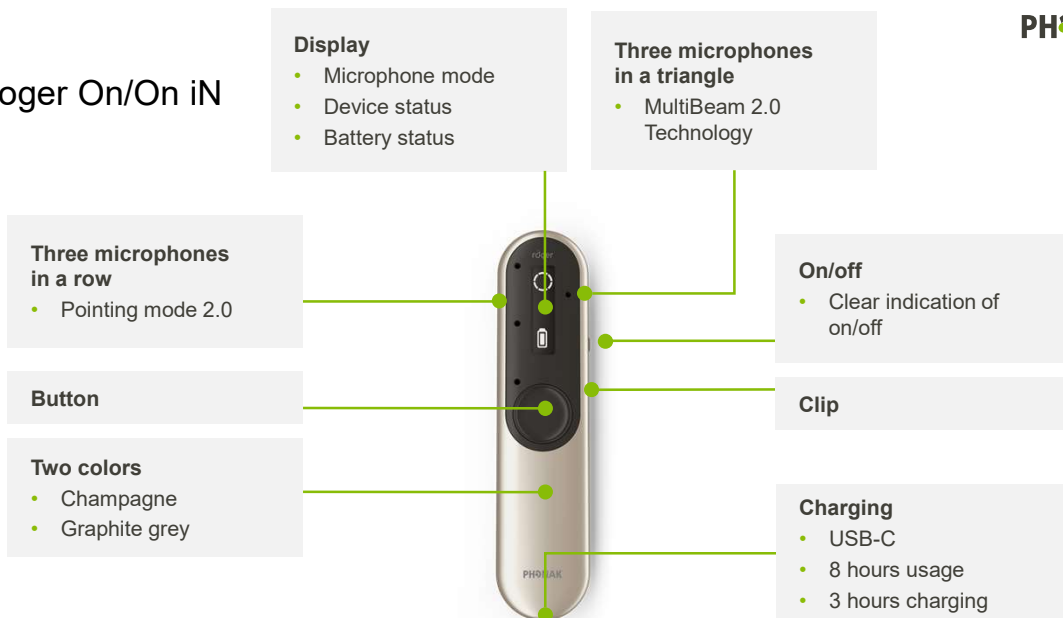

36

Automatic switching to the situation



Presenter mode
with orientation-independent in
presenter mode



Pointing mode 2.0
with three microphones



Table mode
with MultiBeam 2.0 Technology



37

Streaming TV and Multimedia



38



Automation

- Allows for appropriate setting(s) to be chosen, automatically
 - Noise-cleaning features
 - Beamforming technology
 - Roger/Remote Mic

39



Together, we
change lives

40